# Amplicord®

LARYNGOPHONE • ELECTRO LARYNX

# Amplicord®

MANUEL • MANUAL

## 1

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*The design of the **Amplicord mod. 55-X** is the result of previous experience and incorporates a fully integrated circuit. The whole of the outer casing is in PVC to ensure strongness, safeness and ease of use.*

### Features

**Size :** 31.6 mm diameter by 113 mm long

**Weight :** 87 gr. (without batteries)

**Storage battery :** tension 220 ACV – 50 Hz

**Power supply :** rechargeable battery Ni-Mh 9V – 150 mAh

**Recharging time :** 14 hours

**Amplicord mod. 55-X** will operate for approximately 20 hours of alternate speech at average pitch. If necessary, the storage battery can be replaced by an ordinary commercial 9 V battery. It will not be as powerful or as durable but, for those who usually wear a vocal prosthesis, it can be a very valid alternative. Under these circumstances you may experience some loss of power, as the power generated from a standard 9-volt is less than what you will experience with the Amplicord rechargeable battery. **Amplicord mod. 55-X** is supplied with battery-charger, rechargeable battery and case.

### How to Use

Switch on and then press the device lightly and evenly onto the neck. The device will start to work automatically. To obtain the best results from your Amplicord speech device, experiment until you find the most satisfactory pitch. This is done by placing the appliance next to the upper part of the throat and pronouncing A-E-I-O-U until the desired sound is obtained. When using **Amplicord mod. 55-X**, it is important that its head is placed evenly against the throat to prevent external noise from disturbing the voice. When using for the first time, it is advisable to carry out a series of exercises, alternating short pauses as one would hear from a natural larynx.

### Guarantee

**Amplicord mod. 55-X**, is under guarantee for all its component parts for a period of one year providing it is not mishandled or dropped. **Maintenance :** should be carried out only by the **Distributor**. The device may be cleaned as often as desired by wiping the outside with methylated spirit.